

IMPORTANT NOTICE

Always activate
PAUSE
on the Companion
BEFORE you
attend to the
patient.



- **PAUSE de-activates all alarm triggers for 3 minutes.**
- At the end of 3 minutes, the Companion will beep to confirm it is coming out of PAUSE.
- Press PAUSE again to extend (if required).

If you require a longer **PAUSED** period, you can activate **SLEEP** mode.

- To **activate SLEEP** mode, hold PAUSE and RESET buttons simultaneously until the screen displays 'SL'.
- To **de-activate SLEEP** mode, hold RESET until the screen displays 'ON'.

***N.B. SLEEP MODE DOES NOT AUTO RESET**