

Quick Start Guide

Wireless Display Starter Kit (3 Patient Sets)

Applies to product codes:

R17-0003 | R17-0004 | R17-0005 | R17-0006

System Contents	
Monitor(s)	RG43 Wall mount Display x1 RG41 Wireless Jill x3 RG2 Wireless Companion Monitor x3
Sensor(s) <i>[combination of - totalling 6 devices]</i>	RG49 Wireless Seatgard RG47 Wireless Bedgard RG38/RG59 Wireless Floorgard RG39 Wireless Under-mattress pad
Transmitter(s)	RG9 Wireless Pad Transmitter x6

Sensor Placement

RG47 Bedgard: Place on top of the mattress and under the bed sheet. The pad can be placed anywhere between the shoulders to the hips depending on the condition of the patient and how early you want the alert.

RG39 Under-mattress: Place under the mattress across the width of the bed at the lower back level. On profiling beds make sure it will not entrap the mechanism when the bed is being adjusted.

For further guidance on placement, please refer to the pad positioning flyer.

RG49 Seatgard: Place mat on the relevant seat on top of or under a cushion, giving consideration to the position of the monitor and the least intrusive method of installing the equipment.

RG38/RG59 Floorgard: Place mat at or near to the area to be monitored such as a doorway or side of the bed.

Receiver/Monitor Placement

The wall mount display panel should be placed at a nurse's station or within a main corridor. The Companion monitors should be placed on or at the bedside to give an alarm reset point. The Jill pagers should ideally be worn by nursing staff to give them immediate warning of an event.

General Operation

When an alarm is triggered, it is transmitted to the wall mount display panel and onto the pagers. Alarms can only be reset from the appropriate monitor at the bedside.



Quick Start Guide

Wireless Display Starter Kit (3 Patient Sets)

Follow the steps below to set up your equipment:

- 1.** Pull out the battery tab or insert 6 AA batteries and plug in the wall adapter to power up the RG43 wireless Display Panel.
- 2.** Mount the RG43 wireless Display Panel to a suitable location such as a nurses station or other suitable central location.
- 3.** Remove the battery tab from the Jill pagers or insert 2 AA batteries and switch them on. It is advised to mount the wall brackets for these beside the Display Panel.
- 4.** Set out the patient sets with each monitor plus 2 additional pads. **Make sure sets stay together** i.e. set 1 Companion monitor and set 1 sensor pads should stay together as one complete set.
- 5.** Pull the battery tab out to power up the Companion or insert 3 AA batteries. A long beep will confirm the batteries have been inserted correctly.
- 6.** Fix Companion monitor in the desired location by the patients bedside using the supplied hanging strap or wall bracket.
- 7.** Ensure that the sensor pad is placed on a smooth, flat surface and the pad is not creased or folded in any way. Switch on the transmitter inside the sensor pad pouch.

To test, apply pressure to the sensor. The Companion monitor status light will flash twice, and the set number will briefly show on the wall mount display unit. Release pressure from the mat, the Companion monitor will flash, an audible and visible alert will be triggered on the Display Panel and will be duplicated on the pagers, to confirm that it is resetting correctly, press reset on the Companion monitor, the alarm should clear from all devices.

When you are satisfied with the points above please complete the daily checks sheet provided before putting the system into use.

**Ramblegard Technical Support Helpline:
028 9448 3979**