

Follow the steps below to set-up your fall prevention equipment:

**1.**

**TEST SET:**

- a) Move switch to **VIB+TONE**
- b) Place hand in front of Pre-Sense - a green LED will flash - hold hand there until LED solid (occupied). The pager will flash '**Sens01**' to confirm it is occupied.
- c) Move hand away from the sensor. A red LED will flash on the Pre-Sense unit and the pager will alarm to confirm absence detected.
- d) To reset the alarm, press and hold '**Cancel**' on the pager until the LCD screen displays --- or switch to **OFF** position.

**2.**

**GENERAL OPERATION:**

- a) Ensure pager is switched to '**VIB+TONE**' position and is mounted on wall bracket outside of W.C.
- b) Take patient into W.C. Once the patient is positioned on the toilet, wait until the green LED on the Pre-Sense unit goes solid before exiting the W.C. Sensor is now active.
- c) Stay outside the W.C. door until patient attempts to leave the toilet causing the pager to alarm.
- d) Reset pager and attend to patient.

**3.**

**LOW BATTERY ALERT**

- a) Both pager and Pre-Sense unit will indicate when the battery is low.
- b) Pre-Sense LED will flash red and will cease general operation when low battery is detected. To replace, undo the screws on the back of the device, insert 3 x AA batteries, refit cover and screws (approx 3 months battery life).
- c) Pager LED will continually flash red but will still continue to operate until 'Bat Low' is also displayed on the screen. We recommend that you replace batteries as soon as red LED begins flashing. To replace, undo 1 screw on belt clip and insert 2 x AA batteries (approx. 2 months battery life).

**4.**

**CLEANING**

Clean devices in accordance with general facility cleaning procedures. Do not submerge in any liquid.